



MIXING SUGGESTION CREATION CC - VITA 3D

Vita® 3D Master	Creation CC	Ratio	Opaque	Enamel
1 M1	D-D2 + BD-A	1:1	A1	S-57
1 M2	D-A1 + D-A2	6:1	A1	S-58
2 L1,5	D-B1 + D-C3	6:1	C1	S-58
2 L2,5	D-D4 + D-B2	2:1	B2	S-58
2 M1	BD-A + AD-2 + TI-5	4:1:1	B1	S-59
2 M2	D-D2 + D-D4	3:2	D4	S-59
2 M3	D-B2 + D-B3 + D-D4	2:1:1	D4	S-59
2 R1,5	D-C1 + HT-53 + TI-5	8:1/2:1/2	A1	S-59
2 R2,5	D-A2 + D-C3	8:1	A2	S-59
3 L1,5	D-A1 + D-C3	1:2	D4	S-59
3 L2,5	D-B3 + D-D4	5:2	B3	S-60
3 M1	D-C1 + HT-53 + TI-5	8:1:2	D3	S-59
3 M2	D-D3 + D-A3,5	5:1	D3	S-60
3 M3	D-B3 + HT-53 + HT-54	5:1:1	B3	S-60
3 R1,5	D-C1 + HT-53 + TI-5	3:2:1/2	D3	S-59
3 R2,5	D-A3,5 + D-C3	5:2	A3	S-60
4 L1,5	D-C3 + HT-53	5:1	D3	S-60
4 L2,5	D-A4 + D-C4 + HT-54	5:2:1	C4	S-60
4 M1	D-C1 + HT-55 + TI-5	4:1:2	A2	S-60
4 M2	D-A3,5 + HT-55 + TI-5	3:2:2	D2	S-60
4 M3	D-A4 + HT-54	5:1	A4	S-60
4 R1,5	D-D3 + HT-55 + TI-5	2:1:2	A2	S-60
4 R2,5	D-A4 + HT-53	4:2	A4	S-60
5 M1	OD-D2 + TI-5 + HT-55	2:2:2	A4	S-60
5 M2	D-A3 + D-C4 + HT-55	1:4:2	A4	S-60
5 M3	D-A4 + HT-55 + HT-54	3:1:2	A4	S-60

The results of this mixing chart relate to defined samples produced in-house and the measuring instruments located there.

No responsibility is taken for the correctness of this information

Vita® is a registered trademark of VITA Zahnfabrik H. Rauter GmbH & Co. KG, Bad Säckingen, Germany.